



Comfort, Health and Style!

JOBST® JOVIPAK®

EFFECTIVE AND COMFORTABLE
NIGHTTIME LYMPHEDEMA GARMENTS



THERAPIES. HAND IN HAND.

www.bsnmedical.com

BSN medical

an Essity company

CONTINUUM OF CARE WITH NIGHTTIME COMPRESSION GARMENTS

HELPING PATIENTS MANAGE LYMPHEDEMA INDEPENDENTLY

Patients with lymphedema may experience fluctuations in swelling overnight, which are associated with discomfort and restless sleep. While compression bandages represent an effective solution to manage edema during the night, self-bandaging requires practice and can be demanding for some patients.

Easy to don and care for, **JOBST® JoViPak®** garments help empower patients to more easily gain control over edema at night.



JOBST® & JOVIPAK®

PROVIDE BEST-IN-CLASS THERAPY SOLUTIONS FOR LYMPHEDEMA

The management of lymphedema can be challenging, both for patients and therapists. **BSN medical** offers several customized solutions to meet the individual's needs. Optimal therapeutic results can be achieved from using **JOBST®** medical compression garments, bandages and wraps, and skin and wound care products.

With over 60 years of expertise in compression therapy and its patient-centered philosophy, **JOBST®** products are the gold standard in lymphedema management.

With the integration of **JoViPak®** products into the **JOBST®** portfolio, **JOBST®** is providing additional effective 'tools' to help patients live their lives as independently as possible.

JoViPak® products were developed by renowned lymphedema therapist, educator, and cancer survivor, JoAnn Rovig. They are used in the management of edema and lymphedema, and are complimentary devices used in wound care, post-surgical procedures, and sports therapy for edema.



FITTED

TO YOUR THERAPY
TO YOUR LIFESTYLE
TO YOUR EVERY DAY AND NIGHT

FOR LIFE

TOTAL BODY SOLUTIONS

COMPRESSION WHERE YOU NEED IT WITH JOBST® JOVIPAK®

JOBST® JoViPak® garments offer easy-to-use alternatives to nighttime compression bandaging. These foam-filled garments provide pressure into the tissue and, with the recommended **JoViJacket™**, provide additional compression for optimal therapeutic results.

Head and Neck

- Standard & extended Chin Strap
- Half & full Face Mask
- Neck Pads
- Eye Pad

Torso

- Shoulder-Torso Sleeve
- Vest
- Busti

Genital area

- Boxer
- Boxer Capri
- Female & Male Genital Pads
- Custom Leg (DG1)


Upper Extremities

- Classic Arm Sleeve
- Inna Sleeve
- Two-Piece Arm Sleeve
- Stitched Finger Glove/ Palm Pad
- Combi Arm Sleeve (daytime garment)
- Box Finger Glove & Pad

Lower extremities

- Classic Full Leg
- Classic Lower Leg
- Innaboot Full Leg & Lower Leg
- Hip-Hugger
- Custom Leg (AF1)
- ADVI
- Binky Boot (for infants and small children)

JOBST® JoViPads and JOBST® PitPaks can be used in therapy on any area of the body.

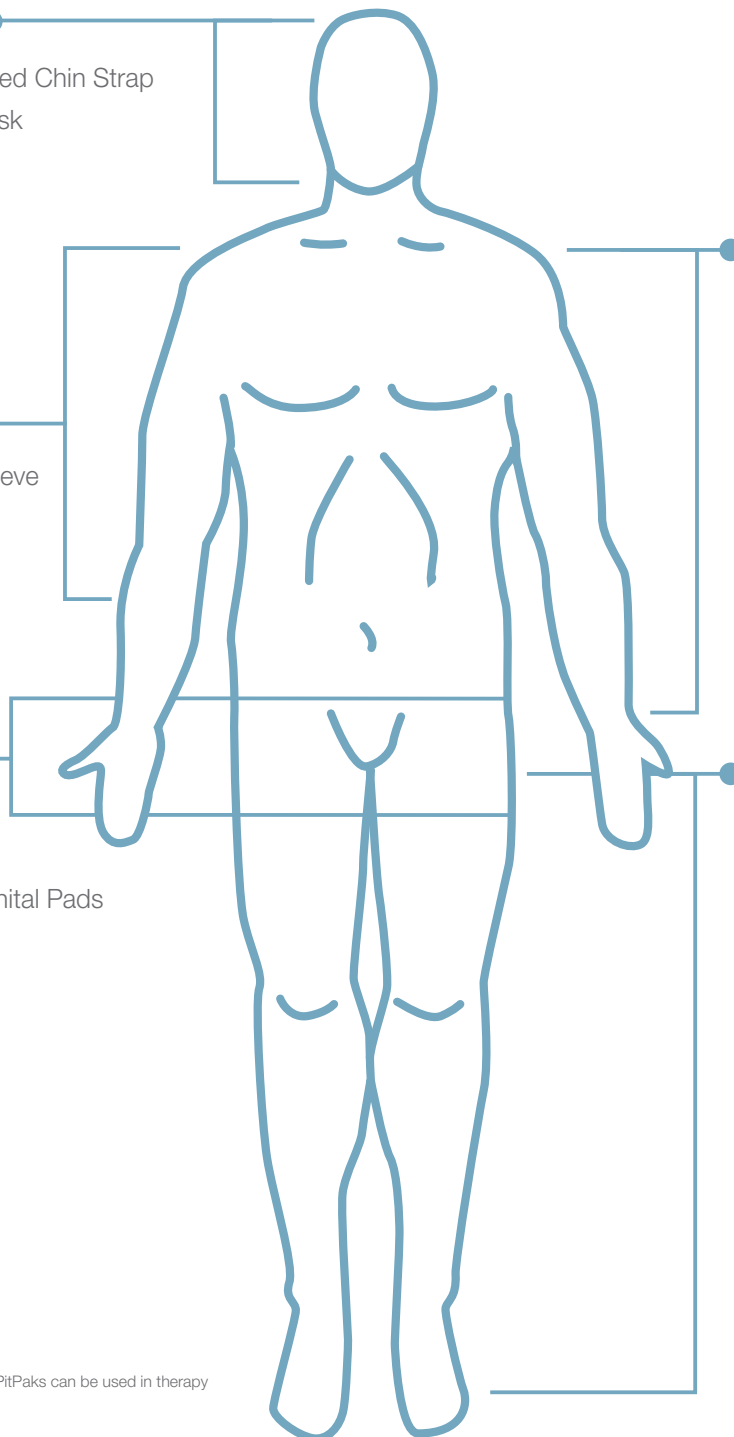


UNIQUE FEATURES AND BENEFITS

Continuous Flow channeling technology.

- Aims to support lymph fluid drainage

Polartec® Power Dry® fabric offers special microfiber yarns and a unique weave to disperse moisture and heat across a wide surface area.

- Helps quick drying of fabric

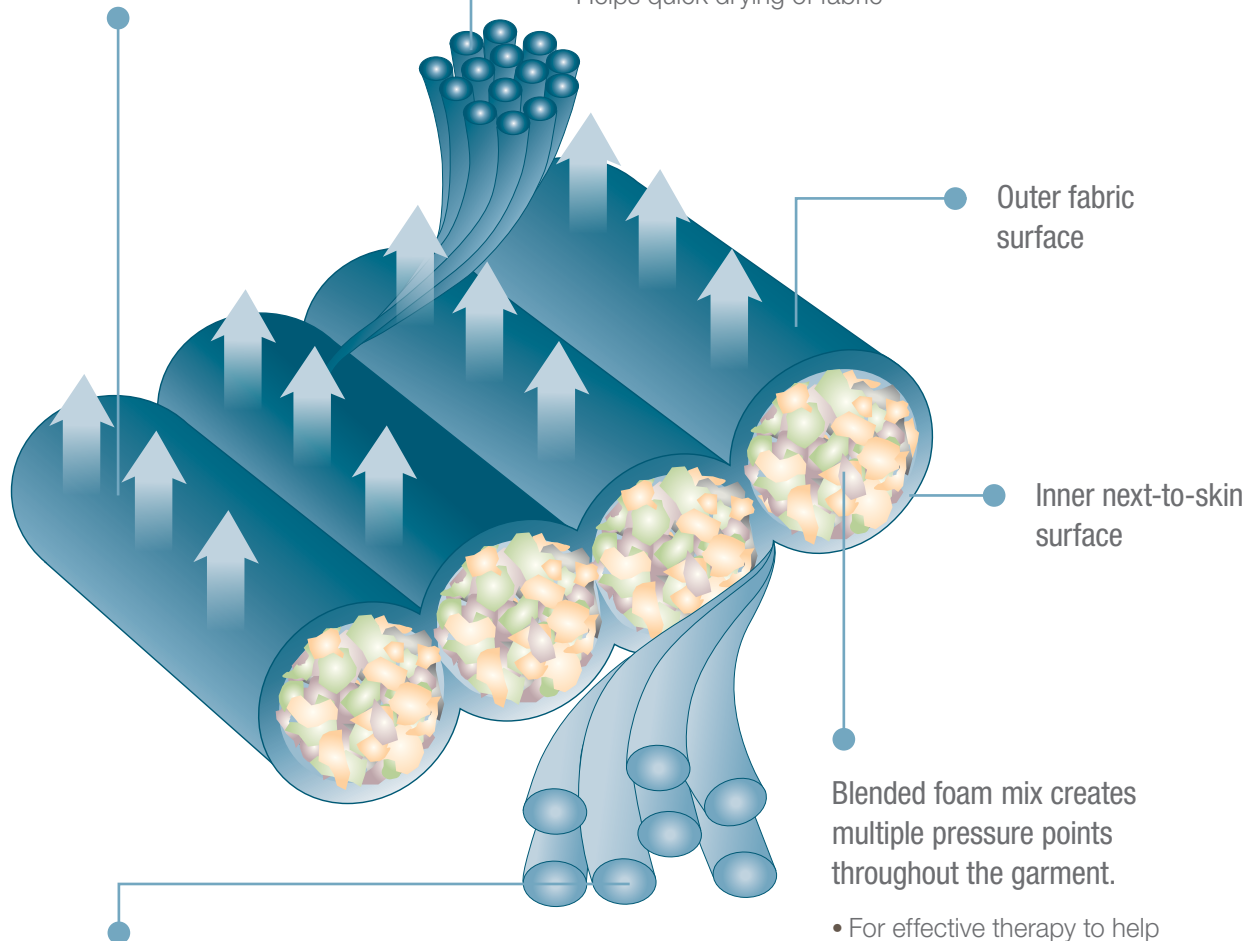

Smooth, highly durable yarns optimized to help move moisture away from the skin.

- For comfortable wearing

Blended foam mix creates multiple pressure points throughout the garment.

- For effective therapy to help improve patient outcomes

The **Polartec® Power Dry®** fabric is also available in **silkweight**, a thinner version which is ideal for low profile slimline garments. **Silkweight** is often recommended for patients having donning and doffing issues.

Garments made of **Organic Cotton/Lycra®** may be a better choice for patients with a skin sensitivity to polyester, and for products with **Velcro®** closures as Velcro® closures will snag the **Polartec® Power Dry®** fabric.


Our JOBST® JoViPak® Classics for Nighttime Compression



JoViJackets:

For extra compression, simply add the **JoViJacket** over your nighttime garment to optimize results.


- Available for Arm, Leg, Busti, Shoulder-Torso, Hip-Hugger, and Boxer

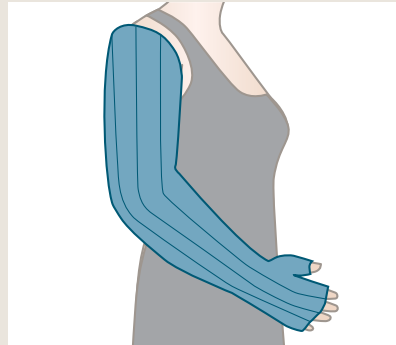

Classic Arm Sleeve:

Designed for lymphedema of the arm following mastectomy or lumpectomy. It encourages lymph drainage to alternate functional pathways.

- Contoured arm sleeve
- Added comfort through anatomically-positioned thumb stub and anticubital dart
- Available in Custom Made in a variety of styles and colors and with optional zipper
- Also available in Ready-to-Wear

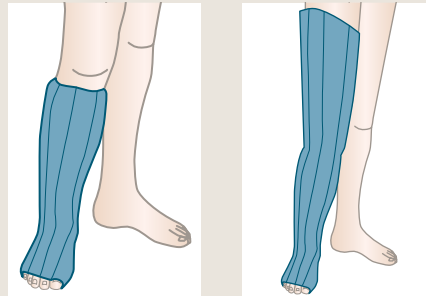

Classic Lower and Full Leg

These garments are suggested for lower / full leg lymphedema or edema and encourage effective lymph drainage.

- Standard version: top of toes exposed
- Available in Custom Made in a variety of styles and colors and with optional zipper
- Also available in Ready-to-Wear

Please refer to the product label and / or package insert for full instructions on the safe use of these products.

Lycra® is a registered trademark of Invista. Polartec® and Power Dry® are registered trademarks of Polartec®, LLC. Velcro® is a registered trademark of Velcro USA.

BSN medical Inc. 5825 Carnegie Blvd., Charlotte, NC 28209-4633
Tel. (+1) 704 554 9933 Fax (+1) 800 835 4325 www.jobst-usa.com
To order toll-free: BSN medical (+1) 800 552 1157

62762 R2 ©2017 BSN medical Inc. REV 08/17

BSN medical

an Essity company